ROTAID™
AED CABINETS

Rotaid™
Solid Plus
Installation manual



Translations: 

Parts-list

This Rotaid package contains:

- Installation manual
- Polycarbonate backplate
- Polycarbonate cover
- Alarm (if ordered)
- Fastening materials

*Screw Set:

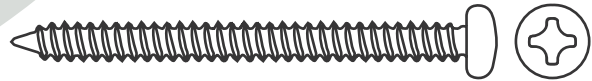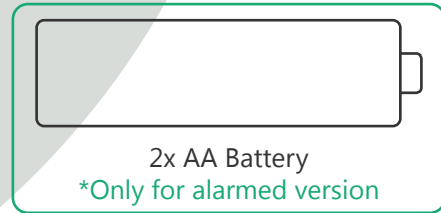4x Tapping screw

1x Bolt

1x Nut

4x Plug

2x DIN screw

Index

pag. 4		English instructions
pag. 5		Nederlandse instructies
pag. 6		Instructions en Français
pag. 7		Deutsche Anleitung
pag. 8		Instrucciones en Español
pag. 9		Istruzioni Italiano
pag. 10		Instrukcja po Polsku
pag. 11		Preparing the alarm
pag. 12		User manual

Rotaid AED Cabinets contact information

Keizersveld 40
5803 AN, Venray
The Netherlands

www.rotaid.com
support@rotaid.com
+31 (0)478 769 052

ROTAID SOLID PLUS

Installation manual

1. Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

2. Mark the 4 holes for drilling and use the screws and rings for fixation.

3. Use an Allen key (5mm) to firmly attach the nylon belt to the back plate.


4. After attaching the belt and cover, now please mount both long bolts in the indicated holes. Mount these by hand via a screwdriver and do not overtighten. You can now hang the AED via its handle on the bolts.

5. You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterwise and rotate to the right until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

Your Rotaid Plus is now ready for use!

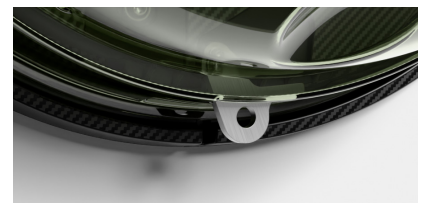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